


Cuspidor-flush Time-delay Kit

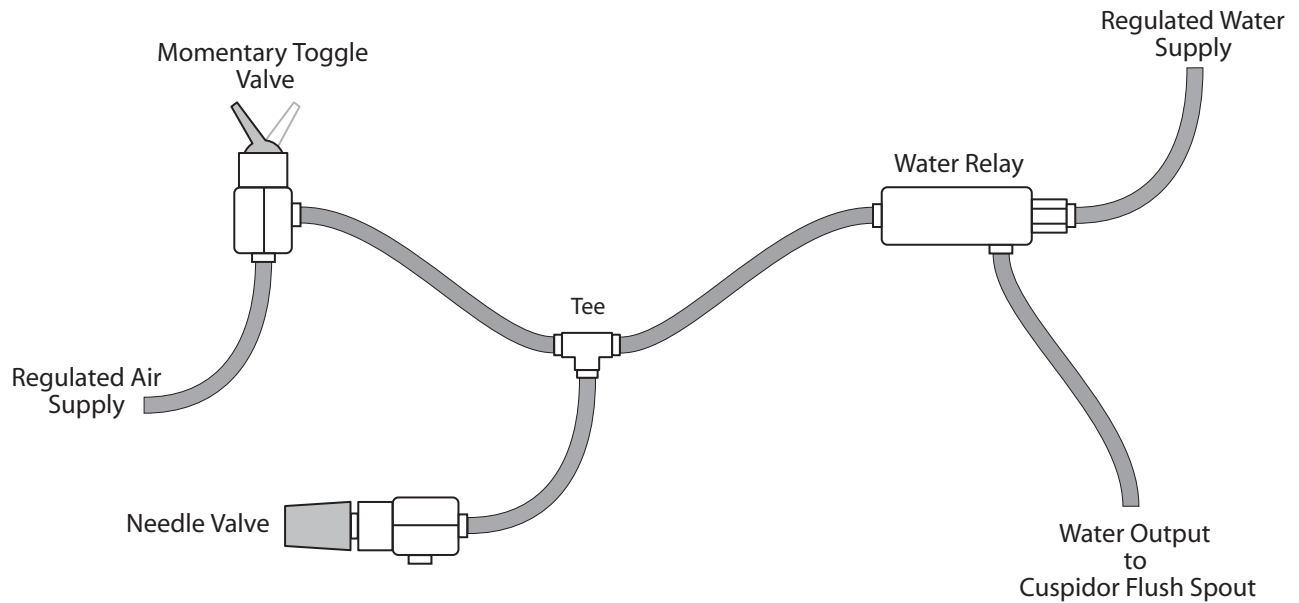

Installation Instructions

- 1) Shut off air and water utilities to the cuspidor and relieve line pressure.
- 2) Install the momentary toggle valve in place of the original flush valve at the cuspidor.
- 3) Hook up the needle valve and water relay as in the diagram above.
- 4) Close the needle valve and open it just barely, about 1/10 of a turn.
- 5) Turn air and water utilities back on and time the duration of the flush circuit when the toggle is actuated.
- 6) Fine tune the setting of the needle valve until the duration of the flush is at the desired level.
- 7) Tuck the tubing and needle valve assembly into the post box under the cuspidor.